

Chromosome Translocations

Invivoscribe Offerings

Translocation Assays



Invivoscribe Portfolio:

- Gel and/or Capillary RUO Assays:
 - *BCL1/JH* Translocation Assay
 - *BCL2/JH* t(14;18) Translocation Assay
 - *BCL2/JH* Translocation Assay
 - *BCR/ABL* t(9;22) Translocation Assay
 - *PML/RAR α* t(15;17) Translocation Assay

- Gel CE-IVD IdentiClone Assays:
 - *BCL1/JH* Translocation Assay
 - *BCL2/JH* Translocation Assay

Table of Contents

Introduction		
Letter to Our Colleagues	2	
Board of Directors	4	
Executive Leadership Team	6	
Best-in-Class Assays & Reagents	8	
LabPMM Clinical Lab Services	10	
		Gel and Capillary - Research Use Only (RUO) Assays 92
		IGH + IGH B-Cell Clonality Assays 94
		IGH Gene Rearrangement Assays 96
		IGH Gene Clonality Assays 98
		IGH Gene Clonality Assays 100
<i>BCL1/JH</i> Translocation Assay	116	
<i>BCL2/JH</i> t(14;18) Translocation Assay	118	
<i>BCL2/JH</i> Translocation Assay	120	
<i>BCR/ABL</i> t(9;22) Translocation Assays	122	
<i>PML/RARα</i> t(15;17) Translocation Assays	124	
LymphoTrack Dx IGH Assays	26	IGH Somatic Hypermutation Assays v2.0 126
LymphoTrack Dx TRG Assays	28	FLT3 Mutation Assays 128
LymphoTrack Dx TRB Assays	30	
LymphoTrack Dx Bioinformatics Software	32	Analyte Specific Reagents
		Analyte Specific Reagents 130
LymphoTrack® - Research Use Only (RUO) Assays	34	Controls, Reagents, and Enzymes
LymphoTrack IGH Leader Somatic Hypermutation Assay	36	Overview 132
LymphoTrack IGH FR1/FR2/FR3 Assays	38	DNA Controls 134
LymphoTrack IGH Assays	42	Low Positive Controls 138
LymphoTrack TRG Assays	44	Internal Controls 139
LymphoTrack TRB Assays	46	RNA Controls 140
LymphoTrack Bioinformatics Software	48	Control Panels 144
		Master Mix Controls 146
LymphoTrack® MRD Solution - Research Use Only (RUO)	50	Custom Products
LymphoTrack Minimal Residual Disease (MRD) Software	52	Custom Products 148
LymphoTrack B-cell & T-cell Low Positive Controls	54	
LymphoQuant B-cell & T-cell Internal Control	55	Reference
		Clonality Testing Workflow Methods 151
Gel and Capillary		Illumina® MiSeq® Workflow Summary 152
IdentiClone® Assays	56	Thermo Fisher Scientific® Ion S5™ PGM™ Workflow Summary 153
IGH + IGH B-Cell Clonality Assays	58	Next-Generation Sequencing Menu 154
IGH Gene Clonality Assays	60	Gel and Capillary Electrophoresis Menu 155
IGH Gene Clonality Assays	62	Common Technical Support Questions 156
IGH Gene Clonality Assays	64	Recent Poster Abstracts 160
TCRB + TCRG T-Cell Clonality Assays	66	
TCRB Gene Clonality Assays	68	Product List
T-Cell Receptor Gamma Gene Rearrangement Assay 2.0	70	<i>BCL1/JH</i> Translocation Assay 76
		<i>BCL2/JH</i> Translocation Assay 78
		FLT3 Mutation Assay - Gel Detection 82
		FLT3 Mutation Assay 2.0 - ABI Fluorescence Detection 84
		CDx FLT3 Mutation Assay (CE-marked) 86
		CDx FLT3 Mutation Assay - AUS 88
		CDx FLT3 Mutation Assay - Japan 90

Invivoscribe RUO Translocation Assays



Translocation	Assay Description
BCL1/JH	Agarose Gel detection
BCL2/JH	Agarose Gel detection
BCL2/JH t(14;18)	Agarose Gel detection
BCR/ABL t(9;22)	Available in Gel and ABI formats RNA-based qualitative <u>nested PCR</u> assay
PML/RaRa t(15;17)	Available in Gel and ABI formats RNA based qualitative <u>nested-PCR</u> assay



Invivoscribe CE-IVD Translocation Assays

Translocation	Associated Diseases	Assay Description
BCL1/JH	Mantle Cell Lymphoma, Multiple Myeloma other Plasma Cell Neoplasms	Agarose Gel detection CE-IVD Sensitivity: up to 10⁻⁴
BCL2/JH	Follicular Lymphoma, Diffuse large B-Cell Lymphoma	Agarose Gel detection CE-IVD Sensitivity: up to 10⁻³



Translocation Assays

BCL1/JH Translocation Assay





Product Offerings:

- **IdentiClone® BCL1/JH Translocation Assays**
- Concordance of 88% positive samples of *BCL1/JH* translocations
- Breakpoints outside of the *CCND1/MTC* locus will not be identified by this particular test.
- Specificity of 100% and Sensitivity up to **10⁻⁴**
- The IdentiClone® *BCL1/JH* Translocation Assay was designed by the EuroClonality group as part of the BIOMED-2 concerted action¹

Available as **gel detection assay**

¹Van Dongen, JJM *et al.* Design and standardization of PCR primers and protocols for detection of clonal immunoglobulin and T-cell receptor gene recombinations in suspect lymphoproliferations: Report of the BIOMED-2 Concerted Action BMH4-CT98-3936. *Leukemia*. 2003, **17(12)**:2257-2317

IdentiClone[®] *BCL1/JH* Translocation Assay

Common sources of genomic DNA:

- 5 mL of peripheral blood, bone marrow biopsy, or bone marrow aspirate anti-coagulated with heparin or EDTA
- Minimum 5 mm cube of tissue
- 2 µg of genomic DNA
- Formalin-fixed paraffin embedded tissue or slides



Product Offerings:

- **BCL1/JH Translocation Assay**
- Breakpoints outside of the CCND1/MTC locus will not be identified by this particular test.
- The BCL1/JH Translocation Assay was designed by the EuroClonality group as part of the BIOMED-2 concerted action¹

Available as **gel detection assay**

¹Van Dongen, JJM *et al.* Design and standardization of PCR primers and protocols for detection of clonal immunoglobulin and T-cell receptor gene recombinations in suspect lymphoproliferations: Report of the BIOMED-2 Concerted Action BMH4-CT98-3936. *Leukemia*. 2003, **17(12)**:2257-2317


BCL1/JH Assays - Kit Contents



- Robust assay, which targets the Major Translocation Cluster (MTC) Region
- **BIOMED-2 design**, 33/330 reactions, 4 tubes (pos./neg. controls+MM+SCSL)



BCL1 MTC primer + JH primer

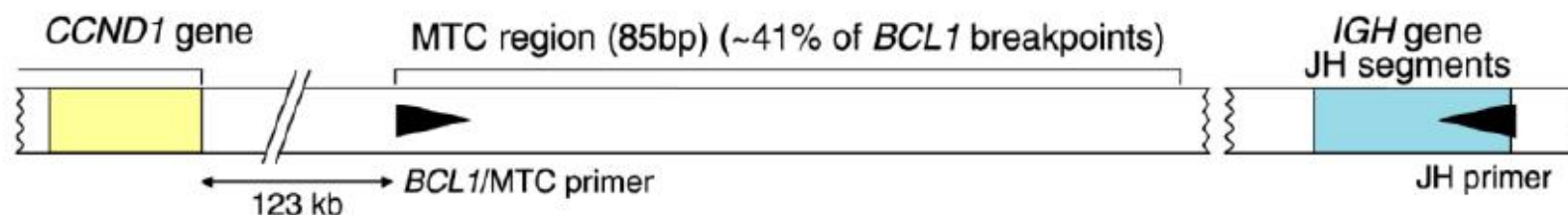

MM = Master Mix
SCSL = Specimen Control Size Ladder

BCL1/JH Translocation Assay




Gel Electrophoresis Detection:

- 2% Agarose Gel
- **One or more prominent positive** bands within the valid size range should be considered as positive for the detection of *BCL1/JH* translocation

BCL1/JH Tube

Lane 1 = 100% IVS-0010
Lane 2 = 10% IVS-0010
Lane 3 = 1% IVS-0010
Lane 4 = 0.1% IVS-0010
Lane 5 = 100% IVS-0000

Valid Size Range =
150-2000 bp

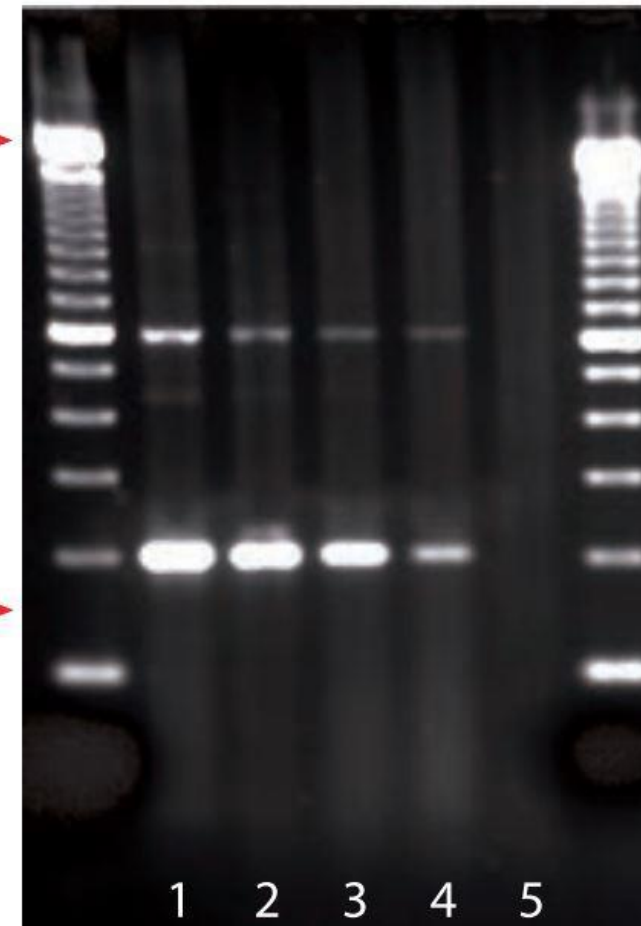

Translocation Assays

BCL2/JH Translocation Assay





Product Offerings:

- **IdentiClone® BCL2/JH Translocation Assays**
- **2% Agarose** gel detection
- Detects about **88%** of BCL2/JH translocations¹
- CE-IVD Sensitivity: up to **10⁻³**
- This IdentiClone® BCL2/JH Translocation Assay was found to be more sensitive than Southern blot analysis
- The IdentiClone® BCL2/JH Translocation Assay was designed by the EuroClonality group as part of the BIOMED-2 concerted action¹

Available as **gel detection assay**

¹Van Dongen, JJM *et al.* Design and standardization of PCR primers and protocols for detection of clonal immunoglobulin and T-cell receptor gene recombinations in suspect lymphoproliferations: Report of the BIOMED-2 Concerted Action BMH4-CT98-3936. *Leukemia*. 2003, **17(12)**:2257-2317

Common sources of genomic DNA:

- 5ml of peripheral blood, bone marrow biopsy, or bone marrow aspirate anti-coagulated with heparin or EDTA
- Minimum 5mm cube of tissue
- 2µg of genomic DNA
- Formalin-fixed paraffin embedded tissue or slides



Product Offerings:

- **BCL2/JH Translocation Assay**
- **2% Agarose** gel detection
- Was optimized using positive and negative control samples with multiplex master mixes
- The *BCL2/JH* Translocation Assay was designed by the EuroClonality group as part of the BIOMED-2 concerted action¹

Available as **gel detection assay**

¹Van Dongen, JJM *et al.* Design and standardization of PCR primers and protocols for detection of clonal immunoglobulin and T-cell receptor gene recombinations in suspect lymphoproliferations: Report of the BIOMED-2 Concerted Action BMH4-CT98-3936. *Leukemia*. 2003, **17(12)**:2257-2317

BCL2/JH Assay - Kit Contents



- **BIOMED-2 design**, 8 tubes (3 pos.+1 neg controls+ 3 breakpoint MM + SCSL)
- Targets **5' Mbr** (major breakpoint region), **3' Mbr**, **mcr** (minor cluster region), and JH regions



Tube A
BCL2 Mbr + IGH JH



Tube B
BCL2 3' Mbr + IGH JH



Tube C
BCL2 mcr + IGH JH



MM = Master Mix
SCSL = Specimen Control Size Ladder

BCL2/JH Translocation Assay




Gel Electrophoresis Detection:

- **2% Agarose Gel**
- **One or more prominent positive bands** within the valid size range should be considered as positive for the detection of *BCL2/JH* t(14;18) translocation

BCL2/JH Tube A

Lane 1 = 100% IVS-0007
Lane 2 = 100% IVS-0030
Lane 3 = 1% IVS-0030
Lane 4 = 100% IVS-0000

Valid Size Range =
100-2500 bp

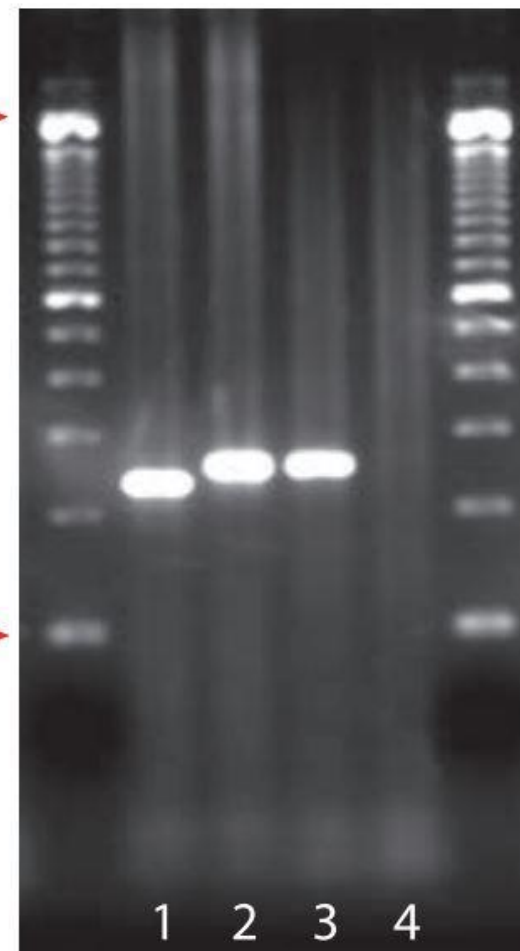

Translocation Assays

BCL2/JH t(14;18) Translocation Assay



BCL2/JH t(14;18) Translocation Assay



Product Offerings:

- Our **BCL2/JH t(14;18) Translocation Assay** identifies *IGH-BCL2* t(14;18)(q32;q21) translocations
- **Agarose Gel** detection
- Can be run either in a standard or nested assay format
- Primers in the *BCL2/JH* master mixes target the joining region of the *IGH* gene and distinct regions of the *BCL2* gene

Specimen Requirements:

- This assay tests genomic DNA as template

Available as **RUO gel detection assay**

BCL2/JH t(14;18) Assay - Kit Contents



- 8 tubes (3 positive controls + 4 breakpoint MM + amplification control)
- Targets **BCL2 Mbr** (major breakpoint region) and **BCL mcr** (minor cluster region), and JH regions



Mix 1a

Outside Mbr primers



Mix 1b

Inside Mbr primers



Mix 2a


Outside mcr primers




Mix 2b

Inside mcr primers

BCL2/IgH Fusion Gene
MBR J_H Breakpoints

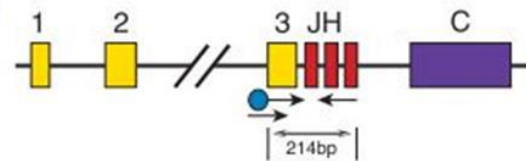

BCL2/IgH Fusion Gene
mcr J_H Breakpoints

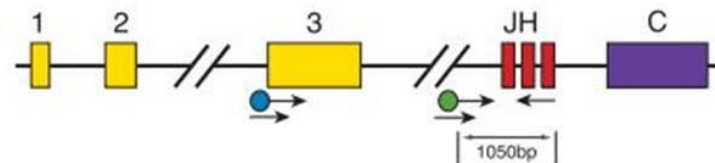

BCL2/JH t(14;18) Translocation Assay




Gel Electrophoresis Detection:

- **2% Agarose Gel**
- **One or more positive bands** within the valid size range should be considered as positive for the detection of *BCL2/JH* translocation

BCL2/JH t(14;18) Mix 1b (Nested)

Lane 1 = 100% IVS-0030
Lane 2 = 10% IVS-0030
Lane 3 = 1% IVS-0030
Lane 4 = 0.1% IVS-0030
Lane 5 = 100% IVS-0000

Valid Size Range =
80-300 bp

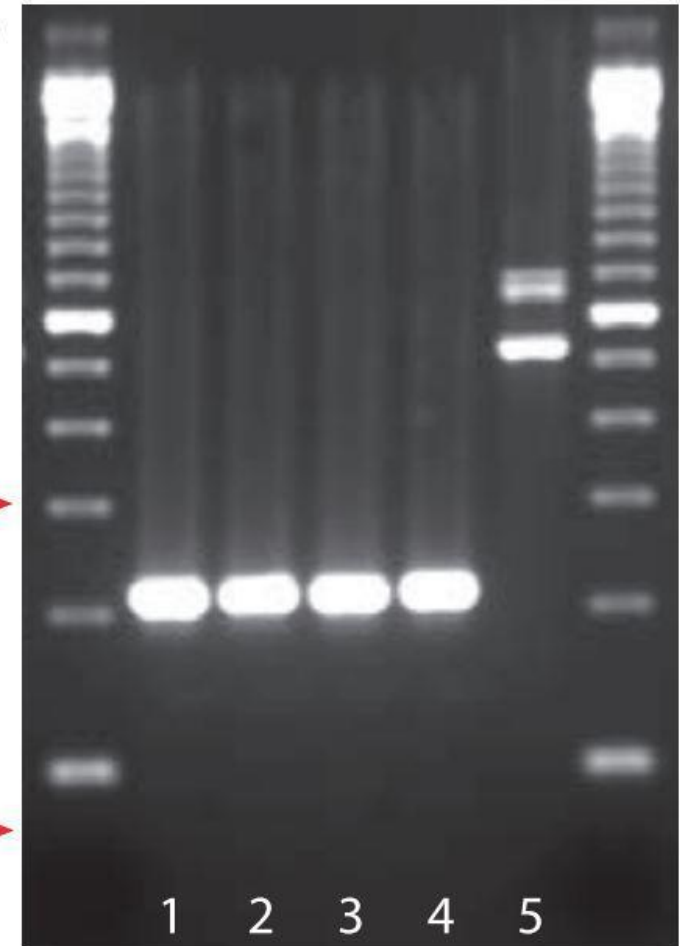

Translocation Assays

BCR/ABL t(9;22) Translocation Assays



BCR/ABL t(9;22) Translocation Assays



Product Offerings:

- **BCR/ABL t(9;22) Translocation Assay**
- **RNA-based** qualitative nested PCR assay
- Molecular detection of **Philadelphia chromosome (Ph1)-positive** cells by nested reverse transcriptase PCR is faster and significantly more sensitive than cytogenetics or other methods
- This assay detects and identifies the variety of p190-, p210-, and p230-type transcripts produced from all known *BCR/ABL* translocations

Specimen Requirements

- This assay tests complementary DNA (cDNA) template

Available as **RUO assay** for **agarose/ polyacrylamide gel** as well as **ABI formats**


BCR/ABL t(9;22) Translocation Assays




- Tests e1a2, e1a3, b2a2, b2a3, b3a2 and b3a3
- Detects control ABL, p190, p210, p230 transcripts
- Both p190 and p210 transcripts encode proteins with aberrant **tyrosine kinase activity**

BCR/ABL t(9;22) Translocation

p210 Type, b3a2 junction

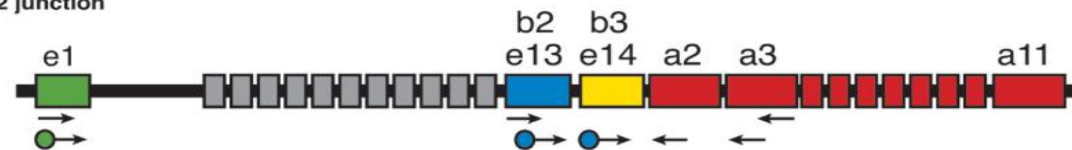

ABL cDNA Control

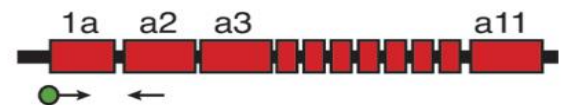


BCR/ABL t(9;22) Translocation Assays




Gel Electrophoresis Detection:

- **2% agarose or 6% polyacrylamide/TBE gel**
- A *BCR-ABL* translocation is indicated if **just one of the 2nd round master mixes** (Mix 2b, Mix 2c, Mix 3b, Mix 3c, or Mix 3d) generates product(s) of the valid size

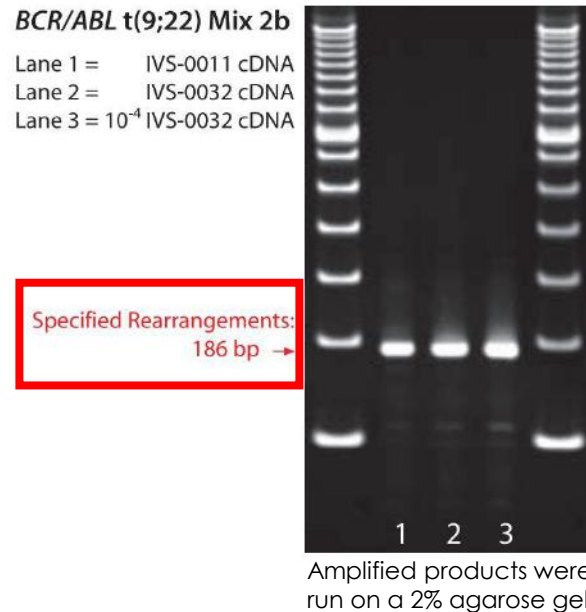

Capillary Electrophoresis Detection (ABI):

- Panel 1: cDNA synthesized from an alternative 100% control RNA
- Panel 2: cDNA synthesized from the recommended 100% clonal control RNA
- Panel 3: cDNA synthesized from a 10⁻⁴ dilution of the recommended clonal control RNA

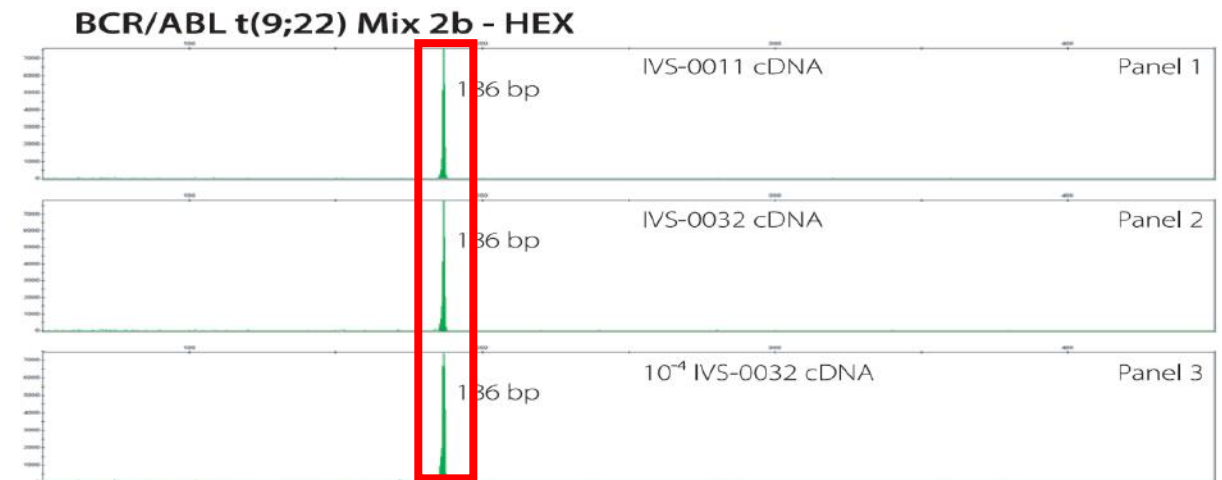

Translocation Assays

PML/RAR α t(15;17) Translocation Assays



PML/RARα t(15;17) Translocation Assays



Product Offerings:

- **PML/RARα t(15;17) Translocation Assays**
- **RNA based** qualitative **nested-PCR** assay
- Detects all three forms of **PML/RARα** translocation; **short (S)**, **long (L)** and **variable (V)** forms

Specimen Requirements

- This assay tests complementary DNA (cDNA) template

Available as **RUO assay** for **agarose/ polyacrylamide gel** as well as **ABI formats**

PML/RAR α t(15;17) Assay - Kit Contents



- 6 tubes (2 RNA controls + 4 MM)
- Targets: RAR α , PML/RAR α , S- & L-forms and L-forms



Mix 1
RAR α



Mix 2a
PML/RAR α




Mix 2b
S- and L-forms




Mix 2c
L-forms


PML Gene
Chromosome 15

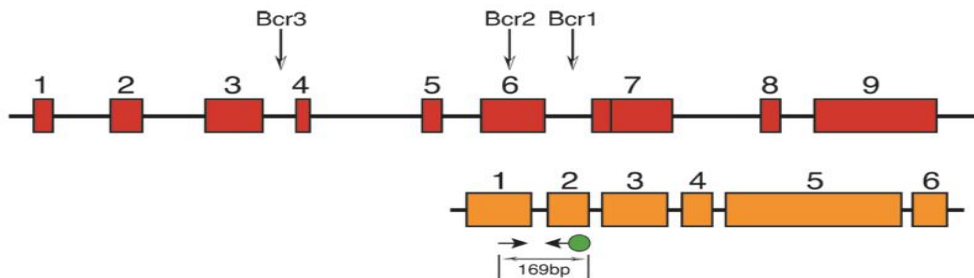

PML/RAR α t(15;17)
S-Form Rearrangement
(Type A Translocation)

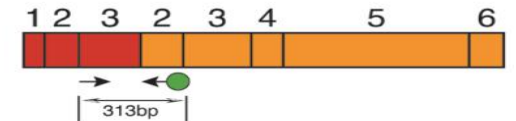

PML/RAR α t(15;17)
L-Form Rearrangement
(Type B Translocation)

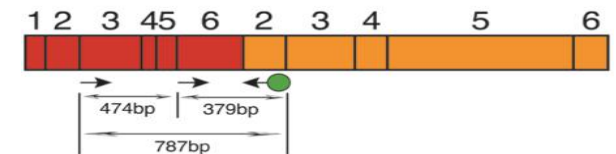


PML/RAR α t(15;17) Translocation Assays




Gel Electrophoresis Detection:

- 2% agarose or 6% Polyacrylamide/TBE Gel
- **One or more positive bands** within the valid size range should be considered as positive for the detection of PML/RAR α translocation

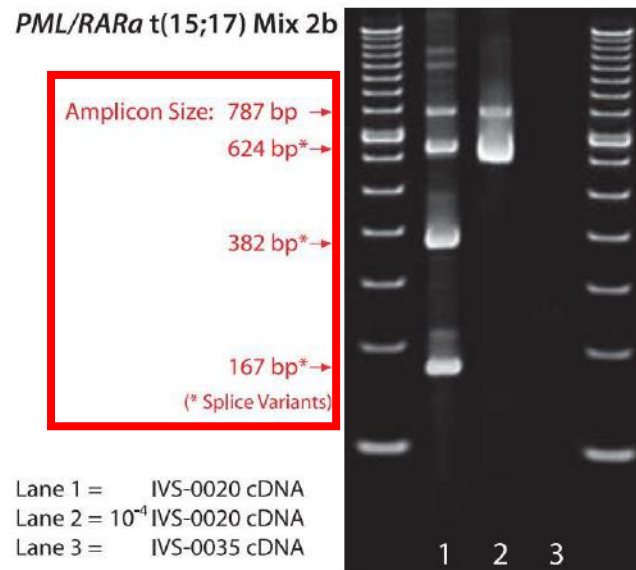

Amplified products were run on a 6% PAGE gel

Capillary Electrophoresis Detection (ABI):

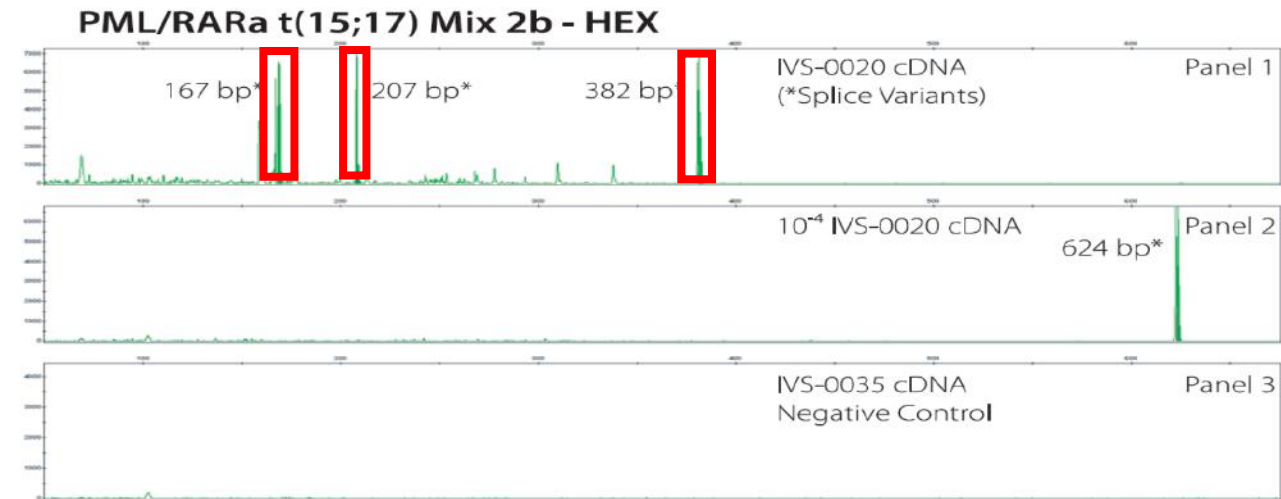

Take Home Message

Translocation Assays





IVS Translocation Assays Strengths

- Molecular identification of genetic markers
- Available for Gel and/or ABI formats
- Efficient and reliable standardized tests, reagents, and controls
- Extensive quality control and quality assurance
- Cutting-edge tools used for improving the quality of healthcare
- IdentiClone[®] products were designed by the EuroClonality group as part of the BIOMED-2 concerted action

Translocation Assays

Quiz





Which Invivoscribe translocation assays are available as CE-IVD products?

- *PML/RAR α*
- *BCL2/JH t(14;18)*
- *BCR/ABL t(9;22)*
- *BCL1/JH*
- *BCL2/JH*



What are key characteristics of a nested PCR?

- Faster TAT
- A first and a second stage of PCR
- Increased LOD



Which Invivoscribe translocation assays are based on the EuroClonality/BIOMED-2 Concerted Action?

- *PML/RAR α* Translocation
- *BCR/ABL t(9;22)*
- *IdentiClone BCL1/JH*
- *IdentiClone BCL2/JH*
- *BCL2/JH t(14;18)*



Which Invivoscribe translocation assays are available for gel and ABI detection?

- *PML/RAR α* Translocation
- *BCR/ABL t(9;22)*
- *IdentiClone BCL1/JH*
- *IdentiClone BCL2/JH*
- *BCL2/JH t(14;18)*



Thank You!

Any questions?